EN: EARSIO – USER MANUAL

Pocket-size Electric Ear Cleaner

USER MANUAL

Thank you for purchasing our product.

PACKAGE INCLUDES:

- 1 x Electric Ear Wax (main unit)
- 2 x Suction Head
- 1 x Cleaning Brush
- 1 x Protective box

SPECIFICATIONS

Material: Plastic, silicone

Power Source: Battery Powered 1 AAA Battery (Not included)

Size: Approx.11.5 X 5.5 X 3cm

Suitable for both kids and adults

WARNINGS

- When used, please do not be touched by other persons or objects, sit down in a safe place.
- When used by children, please operate by the guardian.
- For clearing off wet earwax, please check whether the sucker and filter is blocked.
- If the eardrum is wounded, please do not use the product.
- If the ear is painful or other injuries, please use it after consulting the doctors.
- Only use this device for the purpose its design.

USER INSTRUCTIONS

Note: Turn the machine on, after you hear the sound of turning, slowly put it into ears horizontally with the suction head aligning with the ears. Ear wax removal will gently vibrating to attract the earwax, quickly sucking away the dirt inside of the ear, into the dirt bin of the machine. Also give your ear a better massage.

Get the ear cleaner in the hand horizontally as using. Slightly dip the sucker at the front end into the ear, move back and forth a few times, the earwax can be easily removed.

*The earwax is absorbed at the front of the sucker. When cleaning earwax, the sucker shall be put into into the ear at the proper position.

ATTENTION: in order to ensure no injuries to the ear, please do not di pinto the ear too deep.

CLEANING AND MAINTENANCE

- The filter can be detached and washed by water, the same as the sucker.
- In some occasions, the filter is difficult to be taken down.
- Please note that the other parts except for filter and sucker shall not be washed by water.

INSTRUCTIONS FOR RECYCLING AND DISPOSAL:

This label means that the product cannot be disposed of as other household waste throughout the EU. To prevent potential damage to the environment or human health from uncontrolled waste disposal. Recycle responsibly to promote the sustainable use of material resources. If you want to return a used device, use the drop-off and collection system, or contact the retailer from whom you bought the product. The retailer can accept the product for environmentally safe recycling.

A declaration by the manufacturer that the product complies with the requirements of the applicable EU Directives.

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