



## EN: EASYBREATHE – USER MANUAL

### WHAT'S INCLUDED?

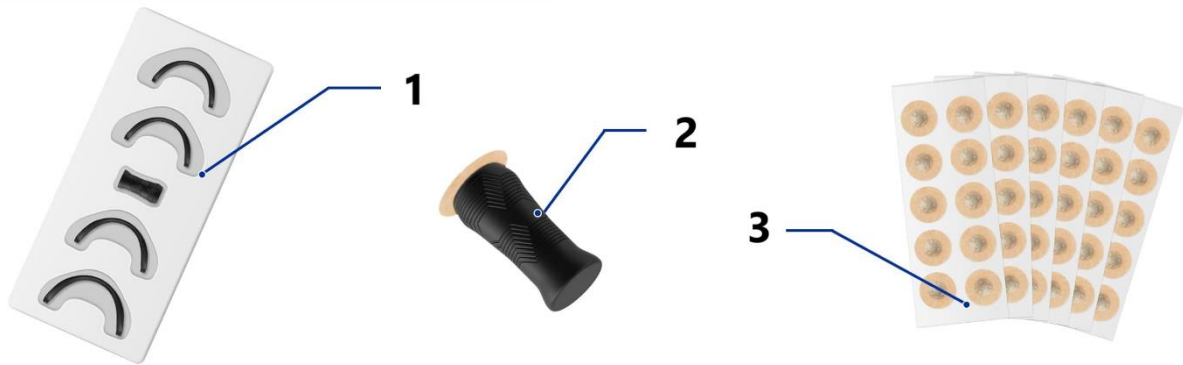

1. **4 NOSE BANDS:** Adjustable nose clips in various sizes to accommodate most users.
2. **1 MAGNETIC APPLICATOR:** Helps secure each nose strip precisely where needed.
3. **15 MAGNETIC TABS:** Adhesive-backed tabs designed for secure attachment of the nose bands.

### INSTRUCTIONS FOR USE

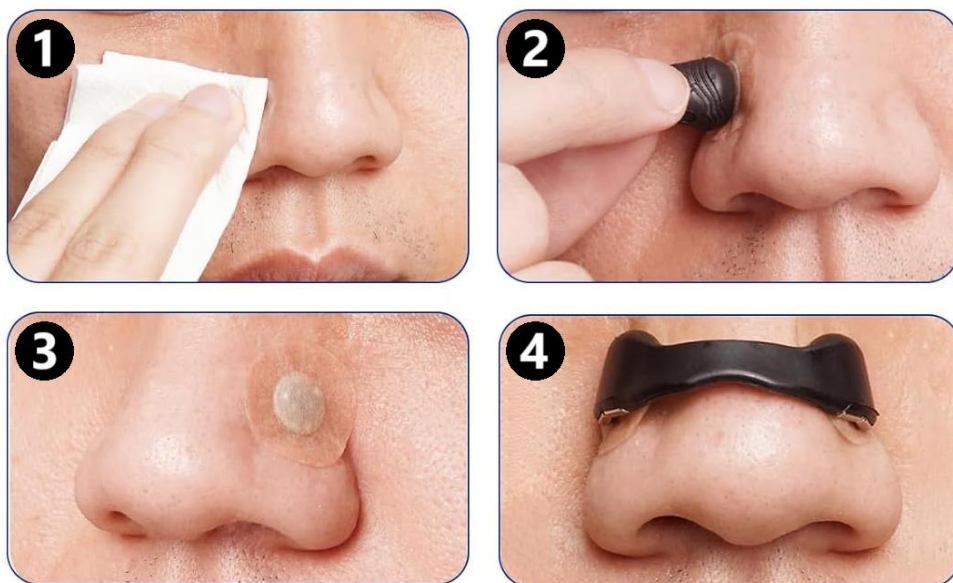

1. **CLEAN:** Wash your nose area to remove dirt and oil. Ensure your skin is completely dry before applying the magnetic nose strips.
2. **APPLY:** Using the applicator, position a magnetic tab on each side of your nose, just above the nostrils. Press firmly for full adhesion.
3. **TAB PLACEMENT:** Place the steel disc slightly above each nostril (not directly on it), ensuring the contoured indent aligns properly.
4. **ATTACH:** Position the EASYBREATHE Band over your nose, allowing the magnetic ends to securely snap onto the tabs.