EN: FIXEDFEET – USER MANUAL

Thank you for purchasing our product. With this non-invasive, non-surgical device, you can alleviate pain in the big toe joint.

PACKAGE CONTENTS

1 pair of toe correctors

PRODUCT SPECIFICATIONS:

Material: nylon, polyurethane

Size: S (36-38) / M (39-42)

WARNINGS

- Do not use the product in any way other than described in the instructions for use, as this may cause damage or injury.
- If the product causes skin irritation, stop using it immediately and consult a doctor.
- Be careful not to let the strap adhere to other clothing, as it can cause damage.

INTENDED USE AND OTHER DETAILS

Gel inserts protect the skin during walking and exercise and act as a cushion to protect the sensitive area of the crooked toe, preventing painful rubbing.

The corrector can be worn discreetly under shoes, socks, and tights as an invisible protection. It can be worn with all types of shoes, including your most elegant pair. The product is intended to be worn directly on the skin.

INSTRUCTIONS:

1. Insert the other toes through the second large opening.
2. Make sure to insert the big toe into the correct part of the corrector (the small section with the opening). Position it so that the back of the foot fits into the large opening.

CARE INSTRUCTIONS

Washing Instructions

- Hand wash below 30 degrees.
- Do not use bleach. Use a neutral detergent that does not contain bleaching ingredients.
- Dry the product in a well-ventilated place away from direct sunlight, i.e., in the shade or indoors. Do not tumble dry.
- Do not iron.
- Not suitable for dry cleaning.

Maintenance Instructions

Do not expose the product to direct sunlight, high temperatures, or other conditions that may damage the product. Store in a dry place with good ventilation.

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